IMMULITE®

TNF α

For use on the IMMULITE®
and IMMULITE® 1000 systems

DPC®

IMMULITE®/IMMULITE® 1000 TNF α

English

Intended Use: For *in vitro* diagnostic use with the IMMULITE and IMMULITE 1000 Analyzers — for the quantitative measurement of Tumor Necrosis Factor α (TNF α) in serum or heparinized plasma, as an aid in the clinical assessment of various immunological diseases.

Catalog Number: **LKNFZ** (50 tests),
LKNF1 (100 tests)

Test Code: **TNA** Color: **Orange**

Summary and Explanation

Tumor necrosis factor α (cachectin) and tumor necrosis factor β (lymphotoxin) are two closely related proteins which share sequence homology of 34% in their amino acid sequence. Both mediators act on their target cells via the same receptors and therefore show similar, but not identical, biological effects.

Under denaturing conditions TNF α is a 17 kDa, non-glycosylated protein. The biologically active form of TNF α is a trimer. Besides this soluble form of TNF α , a 28 kDa membrane-bound form occurs on cell surfaces of TNF-producing cells which may serve as a pool for soluble TNF α and can be proteolytically cleaved from the cell surface.

Different cells are shown to produce TNF α : for example, macrophages, CD4⁺ T cells and NK cells after stimulation with lipopolysaccharides. Additionally, smooth muscle cells, polymorphonuclear neutrophils, astrocytes and a variety of tumor cell lines can produce TNF α .

TNF α acts via two distinct cell surface receptors which are called TNF Receptor I (p55) and TNF Receptor II (p75). These receptors can be identified on virtually all cell types except erythrocytes. Besides the cell-bound forms of the TNF receptors, soluble forms are known which are capable of TNF α binding. They therefore, compete with the cell-bound forms and can inhibit the effects of TNF α .

Due to the occurrence of TNF α receptors on nearly all cells, TNF α shows a wide variety of biological actions. It has cytolytic and cytostatic effects on tumor cells and shows chemotactic activity on neutrophils. TNF α is a growth factor for fibroblasts and stimulates the synthesis of collagenase and prostaglandin E₂ bone resorption can be induced by TNF α because it activates osteoclasts. TNF α enhances the proliferation of T cells after stimulation with IL-2. In the absence of IL-2, TNF α induces the proliferation and differentiation of beta cells.

TNF α serum or plasma levels may be elevated in sepsis, autoimmune diseases, various infectious diseases and transplant rejection.

Principle of the Procedure

IMMULITE/IMMULITE 1000 TNF α is a solid-phase, chemiluminescent immunometric assay.

Incubation Cycles: 1 × 60 minutes.

Specimen Collection

The use of an ultracentrifuge is recommended to clear lipemic samples.

Hemolyzed samples may indicate mistreatment of a specimen before receipt by the laboratory; hence the results should be interpreted with caution.

Centrifuging serum samples before a complete clot forms may result in the presence of fibrin. To prevent erroneous results due to the presence of fibrin, ensure that complete clot formation has taken place prior to centrifugation of samples. Some samples, particularly those from patients receiving anticoagulant therapy, may require increased clotting time.

Blood collection tubes from different manufacturers may yield differing values, depending on materials and additives, including gel or physical barriers, clot activators and/or anticoagulants. IMMULITE/IMMULITE 1000 TNF α has not been tested with all possible variations of tube types. Consult the section on

Alternate Sample Types for details on tubes that have been tested.

Volume Required: 100 μ L serum or heparinized plasma. (Sample cup must contain at least 250 μ L more than the total volume required.)

Storage: 2 days at 2–8°C or 6 months at –20°C.

Warnings and Precautions

For *in vitro* diagnostic use.

Reagents: Store at 2–8°C. Dispose of in accordance with applicable laws.

Follow universal precautions, and handle all components as if capable of transmitting infectious agents. Source materials derived from human blood were tested and found nonreactive for syphilis; for antibodies to HIV 1 and 2; for hepatitis B surface antigen; and for antibodies to hepatitis C.

Sodium azide, at concentrations less than 0.1 g/dL, has been added as a preservative. On disposal, flush with large volumes of water to prevent the buildup of potentially explosive metal azides in lead and copper plumbing.

Chemiluminescent Substrate: Avoid contamination and exposure to direct sunlight. (See insert.)

Water: Use distilled or deionized water.

Materials Supplied

Components are a matched set. The barcode labels are needed for the assay.

TNF α Test Units (LNF1)

Each barcode-labeled unit contains one bead coated with a murine monoclonal anti-TNF α antibody. Stable at 2–8°C until expiration date.

LKNFZ: 50 units. **LKNF1:** 100 units.

Allow the Test Unit bags to come to room temperature before opening. Open by cutting along the top edge, leaving the ziplock ridge intact. Reseal the bags to protect from moisture.

TNF α Reagent Wedge (LNF2)

With barcode. 7.5 mL alkaline phosphatase (bovine calf intestine) conjugated to a rabbit polyclonal anti-TNF α antibody in buffer, with preservative.

Store capped and refrigerated: stable at 2–8°C until expiration date.

Recommended usage is within 30 days after opening when stored as indicated.

LKNFZ: 1 wedge. **LKNF1:** 2 wedges.

TNF α Adjustors (LNFL, LNFH)

Two vials (Low and High) of lyophilized TNF α in a nonhuman serum/buffer matrix, with preservative. Reconstitute each vial with 3.0 mL distilled or deionized water.

Mix by gentle swirling or inversion until the lyophilized material is fully dissolved.

Aliquot and freeze within 3 hours of reconstitution. Stable at –20°C for 6 months after reconstitution.

LKNFZ: 1 set. **LKNF1:** 1 set.

Kit Components Supplied Separately

TNF α Sample Diluent (LNFZ)

For the manual dilution of patient samples.

One vial (25 mL) containing a TNF α -free nonhuman serum/buffer matrix. Stable at 2–8°C for 14 days after opening, or for 6 months (aliquotted) at –20°C.

LSUBX: Chemiluminescent Substrate

LPWS2: Probe Wash Module

LKPM: Probe Cleaning Kit

LCHx-y: Sample Cup Holders (barcoded)

LSCP: Sample Cups (disposable)

LSCC: Sample Cup Caps (optional)

LILCM: IMMULITE Cytokine Control Module (bi-level, human serum-based).

Also Required

Sample transfer pipets, distilled or deionized water, controls.

Assay Procedure

Note that for optimal performance, it is important to perform all routine maintenance procedures as defined in the IMMULITE or IMMULITE 1000 Operator's Manual.

See the IMMULITE or IMMULITE 1000 Operator's Manual for: preparation, setup, dilutions, adjustment, assay and quality control procedures.

Visually inspect each Test Unit for the presence of a bead before loading it onto the system.

Each Sample cup holder can be followed by up to four test units.

Recommended Adjustment Interval:
2 weeks.

Quality Control Samples: Use controls or sample pools with at least two levels (low and high) of TNF α .

Expected Values

Samples from a total of 58 healthy individuals were assayed by the IMMULITE TNF α procedure. The absolute range of these samples was from nondetectable to 8.1 pg/mL.

Consider these limits as *guidelines* only. Each laboratory should establish its own reference ranges.

Limitation

Heterophilic antibodies in human serum can react with the immunoglobulins included in the assay components causing interference with *in vitro* immunoassays. [See Boscato LM, Stuart MC. Heterophilic antibodies: a problem for all immunoassays. Clin Chem 1988;34:27-33.] Samples from patients routinely exposed to animals or animal serum products can demonstrate this type of interference potentially causing an anomalous result. These reagents have been formulated to minimize the risk of interference; however, potential interactions between rare sera and test components can occur. For diagnostic purposes, the results obtained from this assay should always be used in combination with the clinical examination, patient medical history, and other findings.

Performance Data

See Tables and Graphs for data *representative* of the assay's performance. Results are expressed in pg/mL. (Unless otherwise noted, all were generated on serum samples collected in tubes without gel barriers or clot-promoting additives.)

Calibration Range: Up to 1,000 pg/mL (NIBSC 87/650)

Analytical Sensitivity: 1.7 pg/mL

High-dose Hook Effect:
None up to 100,000 pg/mL.

Intraassay Precision (Within-Run):
Statistics were calculated for samples from the results of 20 replicates in a single run. (See "Intraassay Precision" table.)

Interassay Precision (Run-to-Run):
Statistics were calculated for samples assayed in 20 different runs. (See "Interassay Precision" table.)

Linearity: Samples were assayed under various dilutions. (See "Linearity" table for representative data.)

Recovery: Samples spiked 1 to 19 with three TNF α solutions (242, 592 and 988 pg/mL) were assayed. (See "Recovery" table for representative data.)

Specificity: The antibody is highly specific for TNF α . (See "Specificity" table.)

Bilirubin: Presence of bilirubin in concentrations up to 200 mg/L has no effect on results, within the precision of the assay.

Hemolysis: Presence of hemoglobin in concentrations up to 382 mg/dL has no effect on results, within the precision of the assay.

Lipemia: Presence of triglycerides in concentrations up to 3,000 mg/dL has no effect on results, within the precision of the assay.

Alternate Sample Type: Samples (n=10) were collected into plain and heparinized vacutainer tubes.

(Heparin) = 0.94 (Serum) + 10 pg/mL
r = 0.997

Means:
400 pg/mL (Serum)
341 pg/mL (Heparin)

Method Comparison: The assay was compared to the Milenia TNF α assay on 64 patient samples (Concentration range: approximately 28 to 947 pg/mL. See graph.) By linear regression:

(IML) = 0.81 (Milenia) + 40 pg/mL
r = 0.971

Means:
345 pg/mL (IMMULITE)
378 pg/mL (Milenia)

References

- 1) Jacob CO. Tumor necrosis factor α in autoimmunity: pretty girl or old witch? Immunol Today 1992;13:122-5.
- 2) Marano MA, Fong Y, Moldawer LL, Wei H, Calvano SE, Tracey KJ, Barie PS, Manogue K, Cerami A, Shires GT, Lowry SF. Serum cachectin/tumor necrosis factor in critically ill patients with burns correlates with infection and mortality. Surg Gynecol Obstet 1990;170:32-8.
- 3) Maury CPJ,

Teppo AM. Raised serum levels of cachectin/ tumor necrosis factor α in renal allograft rejection. *J Exp Med* 1987;166:1132-7. 4) Braegger CP, Nicholls S, Murch SH, Stephens S, MacDonald TT. Tumor necrosis factor alpha in stool as a marker of intestinal inflammation. *Lancet* 1992;339:89-91. 5) Herbelin A, Nguyen AT, Zingraff J, Urena P, Descamps-Latscha B. Influence of uremia and hemodialysis on circulating interleukin-1 and tumor necrosis factor α . *Kidney Int* 1990;37:116-25. 6) Buck C, Gallati H, Pohlandt F, Bartmann P. Increased levels of tumor necrosis factor α (TNF α) and interleukin 1 β (IL-1 β) in tracheal aspirates of newborns with pneumonia. *Infection* 1994;22:238-41. 7) National Committee for Clinical Laboratory Standards. Procedures for the collection of diagnostic blood specimens by venipuncture; approved standard. 4th ed. NCCLS Document H3-A4, Wayne, PA: NCCLS, 1998.

Technical Assistance

Available outside the United States only. For technical assistance, contact your National Distributor.

Manufactured by EURO/DPC Ltd. under a Quality System registered to ISO 13485:2003.

Tables and Graphs

Intraassay Precision (pg/mL)

| | Mean ¹ | SD ² | CV ³ |
|---|-------------------|-----------------|-----------------|
| 1 | 34 | 1.2 | 3.5% |
| 2 | 327 | 8.6 | 2.6% |
| 3 | 800 | 29 | 3.6% |

Interassay Precision (pg/mL)

| | Mean ¹ | SD ² | CV ³ |
|---|-------------------|-----------------|-----------------|
| 1 | 17 | 1.1 | 6.5% |
| 2 | 33 | 1.9 | 5.8% |
| 3 | 66 | 3.5 | 5.3% |
| 4 | 160 | 8.4 | 5.3% |
| 5 | 322 | 13 | 4.0% |
| 6 | 788 | 35 | 4.4% |

Linearity (pg/mL)

| | Dilution ¹ | Observed ² | Expected ³ | %O/E ⁴ |
|---|-----------------------|-----------------------|-----------------------|-------------------|
| 1 | 8 in 8 ⁵ | 220 | — | — |
| | 4 in 8 | 99 | 110 | 90% |
| | 2 in 8 | 50 | 55 | 91% |
| | 1 in 8 | 25 | 27 | 93% |
| 2 | 8 in 8 | 370 | — | — |
| | 4 in 8 | 180 | 185 | 97% |
| | 2 in 8 | 92 | 93 | 99% |
| | 1 in 8 | 49 | 46 | 106% |
| 3 | 8 in 8 | 571 | — | — |
| | 4 in 8 | 266 | 286 | 93% |
| | 2 in 8 | 151 | 143 | 106% |
| | 1 in 8 | 72 | 71 | 101% |

Recovery (pg/mL)

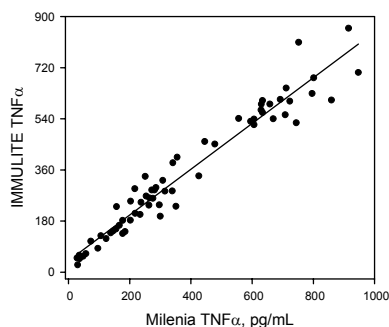
| | Solution ¹ | Observed ² | Expected ³ | %O/E ⁴ |
|---|-----------------------|-----------------------|-----------------------|-------------------|
| 1 | — | 148 | — | — |
| | A | 148 | 153 | 97% |
| | B | 175 | 170 | 103% |
| | C | 212 | 190 | 112% |
| 2 | — | 370 | — | — |
| | A | 345 | 364 | 95% |
| | B | 380 | 381 | 100% |
| | C | 394 | 401 | 98% |
| 3 | — | 571 | — | — |
| | A | 523 | 555 | 94% |
| | B | 526 | 572 | 92% |
| | C | 568 | 592 | 96% |

Specificity

| Compound ¹ | pg/mL Added ² | Apparent pg/mL ³ | % Cross-reactivity ⁴ |
|-----------------------|--------------------------|-----------------------------|---------------------------------|
| IL-1 β | 10,000 | ND | ND |
| IL-2 | 10,000 | ND | ND |
| IL2R | 10,000 | ND | ND |
| IL-4 | 10,000 | ND | ND |
| IL-6 | 10,000 | ND | ND |
| IL-8 | 10,000 | ND | ND |
| IL-10 | 10,000 | ND | ND |
| IL-13 | 10,000 | ND | ND |

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Fabricado pela EURO/DPC Ltd. de acordo com o Sistema de Qualidade registado segundo a norma ISO 13485:2003.
EURO/DPC LTD
Diagnostic Products Corporation
EC REP DPC Biermann GmbH