 IMMULITE[®]
2000

Estradiol

DPC[®]

IMMULITE[®] 2000 Estradiol

English

Intended Use: For *in vitro* diagnostic use with the IMMULITE 2000 Analyzer — for the quantitative measurement of estradiol (estradiol-17 β , E2) in serum, as an aid in the differential diagnosis of amenorrhea, and monitoring of ovulation induction with and without stimulation in assisted reproductive technology (ART).

Catalog Number: **L2KE22** (200 tests),
L2KE26 (600 tests)

Test Code: **E2** Color: **Dark Pink**

CDC Analyte Identifier Code: 1605
CDC Test System Identifier Code: 10418
CLIA Complexity Category: Moderate

Summary and Explanation

Estradiol is a steroid hormone with a molecular mass of 272.3 daltons which circulates predominantly protein-bound. In addition to estradiol, other natural steroidal estrogens include estrone, estriol and their conjugates. Estrogens are hormones secreted principally by the ovarian follicles and also by the adrenals, corpus luteum, placenta and, in males, by the testes.

Estrogenic hormones are secreted at varying rates during the menstrual cycle throughout the period of ovarian activity. The gonadotropins of the anterior pituitary regulate secretion of the ovarian hormones, estradiol and progesterone; hypothalamic control of pituitary gonadotropin production is in turn regulated by plasma concentrations of the estrogens and progesterone. This complex feedback system results in the cyclic phenomenon of ovulation and menstruation.

During pregnancy, the placenta becomes the main source of estrogens. At the menopause, ovarian secretion of estrogens declines at varying rates.

Principle of the Procedure

IMMULITE 2000 Estradiol is a solid-phase, competitive chemiluminescent enzyme immunoassay. The solid phase (bead) is coated with rabbit anti-estradiol polyclonal antibody. The reagent contains

alkaline phosphatase (bovine calf intestine) conjugated to estradiol. The estradiol-enzyme conjugate competes with estradiol in the patient sample for limited antibody-binding sites on the bead. The excess sample and reagent are removed by a centrifugal wash. Finally, chemiluminescent substrate is added to the bead and signal is generated in proportion to the bound enzyme.

Incubation Cycles: 2 \times 30 minutes.

Specimen Collection

Lipemia may interfere with the assay. An ultracentrifuge is recommended to clear lipemic samples.

Hemolyzed samples may indicate mistreatment of a specimen before receipt by the laboratory; hence the results should be interpreted with caution.

Centrifuging serum samples before a complete clot forms may result in the presence of fibrin. To prevent erroneous results due to the presence of fibrin, ensure that complete clot formation has taken place prior to centrifugation of samples. Some samples, particularly those from patients receiving anticoagulant therapy, may require increased clotting time.

Blood collection tubes from different manufacturers may yield differing values, depending on materials and additives, including gel or physical barriers, clot activators and/or anticoagulants. IMMULITE 2000 Estradiol has not been tested with all possible variations of tube types. Consult the section on Alternate Sample Types for details on tubes that have been tested.

Volume Required: 25 μ L serum.

Storage: 2 days at 2–8°C, or
2 months at –20°C.¹⁸

Dilution: It is recommended that patient samples yielding results greater than 1,200 pg/mL be diluted and reassayed. A clinically relevant result is obtained without dilution but the most accurate value is obtained when tested on the sensitive portion of the curve.

Warnings and Precautions

For *in vitro* diagnostic use.

Reagents: Store at 2–8°C. Dispose of in accordance with applicable laws.

Follow universal precautions, and handle all components as if capable of transmitting infectious agents. Source materials derived from human blood were tested and found nonreactive for syphilis; for antibodies to HIV 1 and 2; for hepatitis B surface antigen; and for antibodies to hepatitis C.

Sodium azide, at concentrations less than 0.1 g/dL, has been added as a preservative. On disposal, flush with large volumes of water to prevent the buildup of potentially explosive metal azides in lead and copper plumbing.

Chemiluminescent Substrate: Avoid contamination and exposure to direct sunlight. (See insert.)

Water: Use distilled or deionized water.

Materials Supplied

Components are a matched set. Labels on the inside box are needed for the assay.

Estradiol Bead Pack (L2E212)

With barcode. 200 beads, coated with polyclonal rabbit anti-estradiol. Stable at 2–8°C until expiration date.

L2KE22: 1 pack. **L2KE26:** 3 packs.

Estradiol Reagent Wedge (L2E2A2)

With barcode. 11.5 mL alkaline phosphatase (bovine calf intestine) conjugated to estradiol, in buffer with preservative. Stable at 2–8°C until expiration date.

L2KE22: 1 wedge. **L2KE26:** 3 wedges.

Before use, tear off the top of the label at the perforations, without damaging the barcode. Remove the foil seal from the top of wedge; snap the sliding cover down into the ramps on the reagent lid.

Estradiol Adjustors (LE2L, LE2H)

Two vials (Low and High), 2 mL each, of estradiol in processed human serum, with preservative. Stable at 2–8°C for 30 days after opening, or for 6 months (aliquotted) at –20°C.

L2KE22: 1 set. **L2KE26:** 2 sets.

Before running adjustors, place the appropriate Aliquot Labels (supplied with the kit) on test tubes, so that the barcodes can be read by the on-board reader.

Kit Components Supplied Separately

Estradiol Sample Diluent (L2E2Z)

For the on-board dilution of patient samples. One vial containing 25 mL of concentrated (ready-to-use) processed human serum (with preservative) containing undetectable to low levels of estradiol. Stable at 2–8°C for 30 days after opening, or for 6 months (aliquotted) at –20°C.

L2E2Z: 25 mL.

Barcode labels are provided for use with the diluent. Before use, place an appropriate label on a 16 × 100 mm test tube, so that the barcodes can be read by the on-board reader.

L2E2Z: 3 labels.

L2SUBM: Chemiluminescent Substrate

L2PWSM: Probe Wash

L2KPM: Probe Cleaning Kit

LRXT: Reaction Tubes (disposable)

L2ZT: 250 Sample Diluent Test Tubes (16 × 100 mm)

L2ZC: 250 Sample Diluent Tube Caps

CON6: Tri-level, multi-constituent control

E2TC: Estradiol Control (single level)

Also Required

Distilled or deionized water; test tubes; controls.

Assay Procedure

Note that for optimal performance, it is important to perform all routine maintenance procedures as defined in the IMMULITE 2000 Operator's Manual.

See the IMMULITE 2000 Operator's Manual for: preparation, setup, dilutions, adjustment, assay and quality control procedures.

Recommended Adjustment Interval: 2 weeks.

Quality Control Samples: Use controls or serum pools with at least two levels (low and high) of estradiol.

Expected Values

Based on its relationship to DPC's IMMULITE Estradiol (see Method Comparison), the assay can be expected to have essentially the same reference ranges.

Reference ranges were generated using IMMULITE Estradiol in a multi-national study involving women in apparent good health (age: 16 – 44 years), who volunteered to have blood samples drawn, on a daily basis, throughout one complete ovulatory cycle. (See "Menstrual Cycle Graph.")

| Ovulatory Cycles | n* | Estradiol, pg/mL | | |
|-------------------------------|----------|------------------|---------|-------|
| | | Median | Central | 95% |
| Follicular Phase | 54 (708) | 42 | ND | – 160 |
| Follicular Phase, Days 2 to 3 | 54 (108) | 31 | ND | – 84 |
| Periovulatory, ± 3 days | 54 (378) | 133 | 34 | – 400 |
| Luteal Phase | 54 (604) | 93 | 27 | – 246 |

*Number of subjects (total number of results)

| Ovulatory Cycles | n* | Estradiol, pmol/L | | |
|-------------------------------|----------|-------------------|---------|---------|
| | | Median | Central | 95% |
| Follicular Phase | 54 (708) | 154 | ND | – 587 |
| Follicular Phase, Days 2 to 3 | 54 (108) | 114 | ND | – 308 |
| Periovulatory, ± 3 days | 54 (378) | 489 | 124 | – 1,468 |
| Luteal Phase | 54 (604) | 343 | 101 | – 905 |

*Number of subjects (total number of results)

Another study performed with IMMULITE Estradiol yielded the following results.

| Group | n | Estradiol, pg/mL | | |
|--------------------------|----|------------------|--------|-----------|
| | | Mean | Median | 90% Range |
| Adult males | 50 | 30.5 | 29.7 | ND – 56 |
| Adult females: | | | | |
| Untreated Postmenopausal | 27 | ND | ND | ND – 30 |
| Treated Postmenopausal | 27 | ND | ND | ND – 93 |
| Oral Contraceptives | 61 | 35.2 | 24.5 | ND – 102 |

ND: not detectable.

Estradiol, pmol/L

| Group | n | Mean | Median | 90% |
|--------------------------|----|------|--------|----------|
| | | | | Range |
| Adult males | 50 | 112 | 109 | ND – 206 |
| Adult females: | | | | |
| Untreated Postmenopausal | 27 | ND | ND | ND – 110 |
| Treated Postmenopausal | 27 | ND | ND | ND – 341 |
| Oral Contraceptives | 61 | 129 | 90 | ND – 374 |

ND: not detectable.

Consider these limits as *guidelines* only. Each laboratory should establish its own reference ranges.

Limitations

Pregnancy Samples: Exercise caution when assaying pregnancy samples, since estradiol levels may be high enough to interfere.

Neonatal Samples: The assay has not been validated for use on neonatal samples. Crossreacting steroids, including estradiol, circulating at high concentrations during this period may cause spuriously elevated results.

Heterophilic antibodies in human serum can react with the immunoglobulins included in the assay components causing interference with *in vitro* immunoassays. [See Boscato LM, Stuart MC. Heterophilic antibodies: a problem for all immunoassays. Clin Chem 1988;34:27-33.] Samples from patients routinely exposed to animals or animal serum products can demonstrate this type of interference potentially causing an anomalous result. These reagents have been formulated to minimize the risk of interference; however, potential interactions between rare sera and test components can occur. For diagnostic purposes, the results obtained from this assay should always be used in combination with the clinical examination, patient medical history, and other findings.

Performance Data

See Tables and Graphs for data *representative* of the assay's performance. Results are expressed in pg/mL. (Unless otherwise noted, all were generated on

serum samples collected in tubes without gel barriers or clot-promoting additives.)

Conversion Factor:

pg/mL × 3.671 → pmol/L

Calibration Range: 20 to 2,000 pg/mL (73 – 7,342 pmol/L).

Analytical Sensitivity: 15 pg/mL (55 pmol/L).

Precision: Samples were assayed in duplicate over the course of 20 days, two runs per day, for a total of 40 runs and 80 replicates. (See "Precision" table.)

Precision Profile: In a series of studies involving 10 kit lots, samples spanning the working range were assayed in 10 or 20 replicates per run. (See "Precision Profile" graph.)

Linearity: Samples were assayed under various dilutions. (See "Linearity" table for representative data.)

Recovery: Samples spiked 1 to 19 with three estradiol solutions (800, 1,440 and 4,020 pg/mL) were assayed. (See "Recovery" table for representative data.)

Specificity: The antiserum was generated with estradiol derivative at 6- position, and is highly specific for estradiol. (See "Specificity" table.)

Bilirubin: Presence of bilirubin in concentrations up to 200 mg/L has no effect on results, within the precision of the assay.

Hemolysis: Presence of packed red blood cells in concentrations up to 30 µL/mL has no effect on results, within the precision of the assay.

Lipemia: Presence of triglycerides in concentrations up to 3,000 mg/dL may interfere with the assay. (See "Lipemia" table.)

Alternate Sample Type: To assess the effect of alternate sample types, blood was collected from 33 volunteers into plain, heparinized, and Becton Dickinson SST[®] vacutainer tubes. Equal volumes of the matched samples were spiked with various concentrations of estradiol, to obtain values throughout the calibration range of the assay, and then assayed by the IMMULITE 2000 Estradiol procedure.

(Heparin) = 0.98 (Serum) + 11 pg/mL
r = 0.996

(SST) = 0.98 (Plain Tubes) + 15 pg/mL
r = 0.994

Means:

476 pg/mL (Serum)
477 pg/mL (Heparin)
482 pg/mL (SST)

Method Comparison: The assay was compared to DPC's IMMULITE Estradiol on 97 samples. (Concentration range: approximately 20 to 1,800 pg/mL. See graph.) By linear regression:

(IML 2000) = 0.90 (IML) + 18 pg/mL
r = 0.990

Means:

337 pg/mL (IMMULITE 2000)
354 pg/mL (IMMULITE)

References

- 1) Batzer F. Hormonal evaluation of early pregnancy. *Fertil Steril* 1980;34:1-13.
- 2) Bergquist C, Nillius SJ, Wide L. Human gonadotropin therapy: 1. Serum estradiol and progesterone patterns during conceptual cycles. *Fertil Steril* 1983;39:761-5.
- 3) Erickson GF. Normal ovarian function. *Clin Obstet Gynecol* 1978;21:31-52.
- 4) Garcia JE, Jones GS, Wright GL. Prediction of the time of ovulation. *Fertil Steril* 1981;36:308-15.
- 5) Gautray JP, et al. Clinical investigation of the menstrual cycle: 3. Clinical, endometrial, and endocrine aspects of luteal defect. *Fertil Steril* 1981;35:296-303.
- 6) Goebelsmann U, Mishell DR. The menstrual cycle. In: Mishell DR, Davajan V, editors. *Reproductive endocrinology, infertility and contraception*. Philadelphia: Davis FA, 1979: 67-89.
- 7) Goldstein D, et al. Correlation between estradiol and progesterone in cycles with luteal phase deficiency. *Fertil Steril* 1982;37:348-54.
- 8) Haning RV, et al. Plasma estradiol is superior to ultrasound and urinary estradiol glucuronide as a predictor of ovarian hyperstimulation during induction of ovulation with menotropins. *Fertil Steril* 1983;40:31-6.
- 9) Judd HL, Korenman SG. Effects of aging on reproductive function in women. In: Korenman SG, editor. *Endocrine aspects of aging*. New York: Elsevier Biomedical, 1982:163-97.
- 10) Landgren BM, Aedo AR, Diczfalusy E. Hormonal changes associated with ovulation and luteal function. In: Flamigini C, Givens JR, editors. *The gonadotropins: basic science and clinical aspects in females*. London: Academic Press, 1982:187-201.
- 11) March CM, Goebelsmann U, Nakamura RM, Mishell DR. Roles of estradiol and progesterone in eliciting the midcycle luteinizing hormone and follicle-stimulating hormone surges. *J Clin Endocrinol Metab* 1979;49:507-12.
- 12) Polan ML, et al. Abnormal ovarian cycles as diagnosed by ultrasound and serum estradiol levels. *Fertil Steril* 1982; 37:342-7.
- 13) Radwanska E, et al. Plasma progesterone and oestradiol estimation in the diagnosis and treatment of luteal insufficiency

in menstruating infertile women. *Acta Eur Fert* 1976;39:47. 14) Radwanska E, et al. Correlation between preovulatory serum estradiol and midluteal progesterone levels during induction of ovulation with clomid and HCG. *J Reprod Med* 1980; 24:79-82. 15) Rebar RW, Yen SSC. Endocrine rhythms in gonadotropins and ovarian steroids with reference to reproductive processes. In: Krieger, editor. *Endocrine rhythms*. New York: Raven Press, 1979: 259-98. 16) Robertson RD, et al. Assessment of ovulation by ultrasound and plasma estradiol determinations. *Obstet Gynecol* 1979; 54:686-90. 17) Sileri PK, Febres F. Ovarian hormone synthesis, circulation, and mechanisms of action. In: DeGroot L, et al, editors. *Endocrinology*, vol 3. New York: Grune & Stratton, 1979:1401-17. 18) Burtis CA, Ashwood ER, editors. *Tietz textbook of clinical chemistry*. 2nd ed. Philadelphia: W.B. Saunders, 1994. 19) National Committee for Clinical Laboratory Standards. Procedures for the collection of diagnostic blood specimens by venipuncture; approved standard. 4th ed. NCCLS Document H3-A4, Wayne, PA: NCCLS, 1998.

Technical Assistance

In the United States, contact DPC's Technical Services department.
Tel: 800.372.1782 or 973.927.2828
Fax: 973.927.4101. Outside the United States, contact your National Distributor.

Manufactured by EURO/DPC Ltd. under a Quality System registered to ISO 13485:2003.

Tables and Graphs

Precision (pg/mL)

| | Mean ³ | Within-Run ¹ | | Total ² | |
|---|-------------------|-------------------------|-----------------|--------------------|------|
| | | SD ⁴ | CV ⁵ | SD | CV |
| 1 | 89 | 8.8 | 9.9% | 14 | 16% |
| 2 | 180 | 14 | 7.8% | 19 | 11% |
| 3 | 461 | 20 | 4.3% | 31 | 6.7% |
| 4 | 1,309 | 84 | 6.4% | 102 | 7.8% |
| 5 | 1,800 | 89 | 4.9% | 127 | 7.1% |

Linearity (pg/mL)

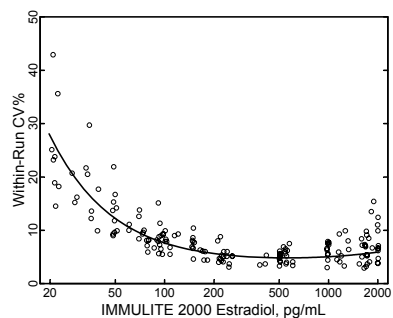
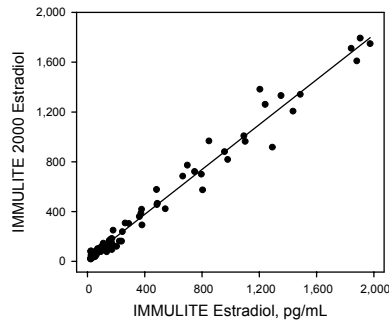
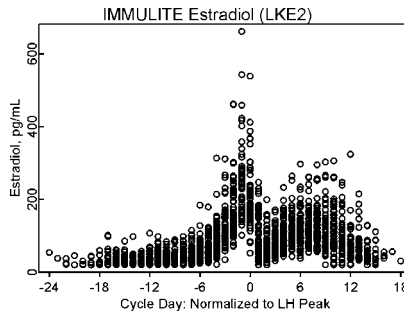
| | Dilution ¹ | Observed ² | Expected ³ | %O/E ⁴ |
|---|-----------------------|-----------------------|-----------------------|-------------------|
| 1 | 8 in 8 ⁵ | 370 | — | — |
| | 4 in 8 | 204 | 185 | 110% |
| | 2 in 8 | 92 | 93 | 99% |
| | 1 in 8 | 43 | 46 | 93% |
| 2 | 8 in 8 | 756 | — | — |
| | 4 in 8 | 394 | 378 | 104% |
| | 2 in 8 | 193 | 189 | 102% |
| | 1 in 8 | 97 | 95 | 102% |
| 3 | 8 in 8 | 1,849 | — | — |
| | 4 in 8 | 975 | 925 | 105% |
| | 2 in 8 | 466 | 462 | 101% |
| | 1 in 8 | 220 | 231 | 95% |

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Fabricado pela EURO/DPC Ltd. de acordo com o Sistema de Qualidade registado segundo a norma ISO 13485:2003.
EURO/DPC LTD
Diagnostic Products Corporation
EC REP DPC Biermann GmbH